

**MICROSALT**

THE HEALTHY EXTRA FOR SAUNA AND INFRARED.



All new and already a prizewinner:  
it was awarded the 2016 PlusX Award.

## THE NEW DIMENSION OF DRY SALT INHALATION.

Great health benefits, simple operation and maintenance as well as a handy format:

SaltProX is the first Microsalt device especially for private customers.

The positive health effects of dry salt have been proven in many scientific studies. Using patented Microsalt technology, we have ensured broad utilisation on the basis of new scientific findings. This innovation received the European Health und Spa Award 2014 as the 'Best Spa Product Innovation Technique'.

With SaltProX, we have now managed to make this unique and patented technology easily available and affordable for private households.

Small, light, handy and elegantly designed – the healthy extra for the sauna or infrared cabin can be harmoniously integrated in any heating cabin with its modern and timeless design. Thanks to its battery operation, Microsalt SaltProX can be used very flexibly, so that it can travel anywhere, even in your hand luggage.



## THE PERFECT COMBINATION FOR YOUR HEALTH.

There are many good reasons for a sauna or infrared cabin. Here's one more: With SaltProX, any sauna or infrared cabin can be converted into a salt inhalation room, because we designed it for use in heating cabins as only here the humidity can be adjusted to less than 40% relative humidity without a problem, so that a pleasant microclimate with a high salt-aerosol concentration is created. This dry salt aerosol is distributed evenly in the cabin. The result: maximum benefit for your health with minimum effort.




Very flexible: Microsalt SaltProX is simply placed on the lounge in the heating cabin or fastened to the wall with the supplied console, and it can easily be removed again at any time.

## ONE PROCEDURE, MANY HEALTH ADVANTAGES: THE MICROSALT CONCEPT.

A stiff breeze blows in your face, water splashes around your feet. The taste of salt on your lips. A visit to the seaside simply feels good. Not just for the soul, but especially the airways.

That is why the Microsalt application is particularly useful for people with respiratory disorders. With its invisible grinding technology, Microsalt SaltProX generates a particularly fine salt aerosol of the highest quality, which then spreads in the entire cabin. Because of the friction of the individual salt crystals, these are crushed to a very fine powder and get also negatively charged

airways and the skin. When inhaling, these fine salt particles – unlike with conventional salt applications – spread across the entire respiratory system, from the nose, the sinuses, the throat to the extreme lung areas, and can have an expectorant effect here. The application of Microsalt results in an in-depth cleaning of the airways right down to the finest alveoli as well as the cleansing and revitalisation of the skin. Regular inhalations can also support the immune system. Microsalt is especially recommended for hay fever, asthma, bronchitis, cystic fibrosis or chronic lung diseases. Microsalt is also suitable for

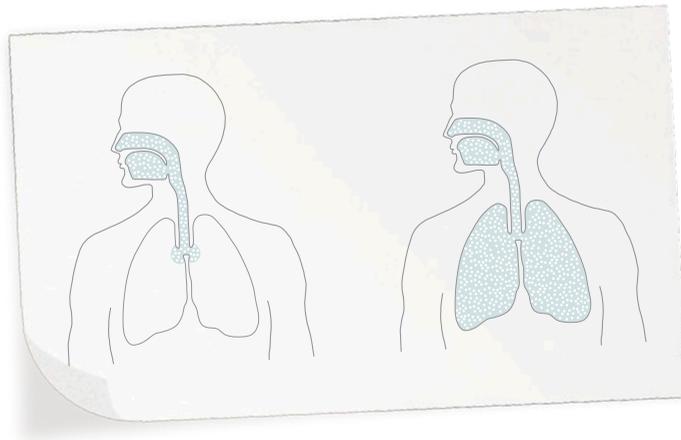

Penetration depth of salt-enhanced air using – on the left – conventional technology and – on the right – Microsalt.

(triboelectric effect). The resulting ionisation of individual salt particles boosts the distribution in the room and thus also the cleaning of the air, that is constantly in touch with the

people frequently suffering from colds or struggling with sinusitis, as well as for people with skin diseases, sleep disorders or depressions.

# AS BENEFICIAL AS A STAY BY THE SEASIDE.

- Dry salt aerosols have strong hygroscopic properties and help dissolve secretions lodged in the airways.
- Dry salt aerosols have an anti-inflammatory and disinfectant effect.
- Dry salt aerosols stimulate the removal of mucus and dirt from the airways.
- With over 99% sodium chloride, the natural rock salt used for the Microsalt application has the highest level of purity found in nature.





## COMFORTABLE AND FOOLPROOF: THIS IS HOW SALTPROX WORKS.

It has never been easier and safer to do something good for your health. The easily understandable function of Microsalt SaltProX is based on the scheme of other products from our company that has been tried and tested a thousand times over. Thus, the operation of the device is comfortable, foolproof and safe. Simply preheat the heating cabin to 30 to 50°C, place the device inside, then switch on the main switch and insert the salt container in the device. After pressing the start/stop button, the device starts with the salt-aerosol production. After about ten minutes, there is

an optimal dry salt concentration in the sauna or infrared cabin, and the colour of the lighting changes from orange to blue. Now you can start with your beneficial, health-boosting inhalation. After an hour, the device will switch off automatically.

The assembly of SaltProX is also child's play: the device is fastened to the wall with the supplied wall holding bracket and can be easily removed again at any time. After about 1.5 hours charging time, the battery will have sufficient charge for three to five applications.



You can see how clever a product is by looking at the details.

Microsalt SaltProX uses specially portioned salt sticks that contain the optimal dose of dry, high-grade salt for one application. The quantity contained in the sticks is designed for the maximum recommended application duration of one hour. To get the device ready for use, simply remove the cup from

the device and insert the content of a salt stick together with the stainless steel crushing ball. Afterwards, just place the cup, made of break-proof and dishwasher-safe special glass, back in the device – done.

This procedure is extremely easy, because the holding bracket automatically guides the salt cup into the right position.



Exact: The salt sticks contain the optimal quantity of salt for one application.



Easy as can be: The holding bracket automatically guides the salt cup in the right position as it is inserted.

# KLAFS

MY SAUNA AND SPA

GERMANY | ERICH-KLAFS-STRASSE 1-3 | D-74523 SCHWÄBISCH HALL  
TELEPHONE +49 (0)791 501-0 | FAX +49 (0)791 501-248 | INFO@KLAFS.COM | WWW.KLAFS.COM

AUSTRIA | SONNWIESENWEG 19 | A-6361 HOPFGARTEN/TYROL  
TELEPHONE +43 (0)5335 23 30-0 | FAX +43 (0)5335 23 30-36 | INFO@KLAFS.AT | WWW.KLAFS.AT

SWITZERLAND | OBERNEUHOFSTRASSE 11 | CH-6340 BAAR  
TELEPHONE +41 (0)41 760 22 42 | FAX +41 (0)41 760 25 35 | BAAR@KLAFS.CH | WWW.KLAFS.CH